TONOVET 1736005**TONOVET Plus 173620, 1736007****Is the TONOVET measurement painless?**

The measurement is painless. A light-weight probe touches the cornea momentarily and very gently, and most patients don't even notice the measurement.

Is the TONOVET measurement result accurate?

Several independent studies prove the accuracy of TONOVET readings. Extensive bench testing and clinical studies have also been performed to ensure the accuracy and repeatability of the measurements.

Why six measurements?

Six measurements are required to provide accurate measurement results by eliminating the variations caused by operator error and heart rate. The TONOVET software will discard the highest and lowest reading, and the final result is an average of the four single measurements.

Can anesthesia still be used?

Icare® TONOVET rebound tonometer is designed to be used without applying topical anesthetic. For most accurate measurements, we recommend not to apply topical anesthetic when measuring with the TONOVET. It is possible to measure an anesthetized eye, but the readings may be slightly affected by the swelling caused by topical anesthetic. However, the affect is most likely insignificant.

Is it possible to load the probe the wrong way?

No, it's not possible to load the probe the wrong way. The mechanical design of the probe and the probe base make it possible to insert the probe only the right way.

Can the probes be re-used?

TONOVET tonometer is designed to use single-use disposable probes. Probes should never be cleaned or sterilized because the process and handling can damage the probe resulting in unreliable measurement results or even damage the tonometer. However, the same probe can be used for measuring the same patient within a reasonably short time period.

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UNITED KINGDOM | 3 Windmill Business Village | Brooklands Close | Sunbury-on-Thames | Surrey | TW16 7DY | UK | T 020 8891 2007 | www.eickemeyer.co.uk

DENMARK | Solbakken 26 | Hammelev | 6500 Vojens | Danmark | T +45 70 20 50 19 | info@eickemeyer.dk | www.eickemeyer.dk

NETHERLANDS | Bellweg 44 | 4104 BJ Culemborg | Nederland | T +31 345 58 9400 | info@eickemeyer.nl | www.eickemeyer.nl

ITALY | Via G. Verdi 8 | 65015 Montesilvano (PE) | Italia | T 085 935 4078 | F 085 935 9471 | info@eickemeyer.it | www.eickemeyer.it

POLAND | Al. Jana Pawła II 27 | 00-867 Warszawa | Polska | T 22 185 55 76 | F 22 185 59 40 | info@eickemeyer.pl | www.eickemeyer.pl

CANADA | 617 Douro Street, Suite #205 | Stratford | Ont. | Canada | N5A 0B5 | T +1 519 273 5558 | F +1 519 271 7114 | info@eickemeyervet.ca | www.eickemeyercanada.ca

TONOVET 1736005**TONOVET Plus 173620, 1736007****Can TONOVET be used when the eye is infected?**

Yes. Just remember that the probe used to measure the infected eye cannot be used again, even for measuring the non-infected eye of the same patient.

Can the same probe be used to measure both eyes of one patient?

Yes, same probe can be used to measure both eyes of one patient if the patient's eyes are healthy. If the patient has an infection in one eye or if there is any doubt that one eye may have a disease that can be transferred from eye to eye, the healthy eye should be measured first. A probe used to measure an infected eye should never be used to measure a healthy eye. If in doubt, always use a new probe.

How will probes be disposed of?

Probes can be disposed of according to the hospital/clinic/practice standard regulations. If such instructions do not exist, place the used probe back to the plastic tube to be disposed of.

Does TONOVET require calibration?

Icare® rebound tonometers do not require any maintenance calibration or regular service. The tonometers don't have any parts that wear out, except for the probe base, which can become dusty or collect some particles that affect the probe movement. The probe base can be changed by the user as described in the user manual. The calibration can be checked if there's doubt about the measurement results. In such case, you should contact your local dealer/distributor.

If the tonometer requires service, for example has been dropped on the floor, you should also contact your local dealer/distributor for further instructions.

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